

All in one slips




Incontinence severity (involuntary loss of urine in 4 hours)



The best thing we have ever developed!

forma-care slip is a very absorbent, anatomically shaped incontinence shaped pad with breathable air TBS outer foil with felt in the hip area. The dermatologically tested and latex free product offers maximum comfort.

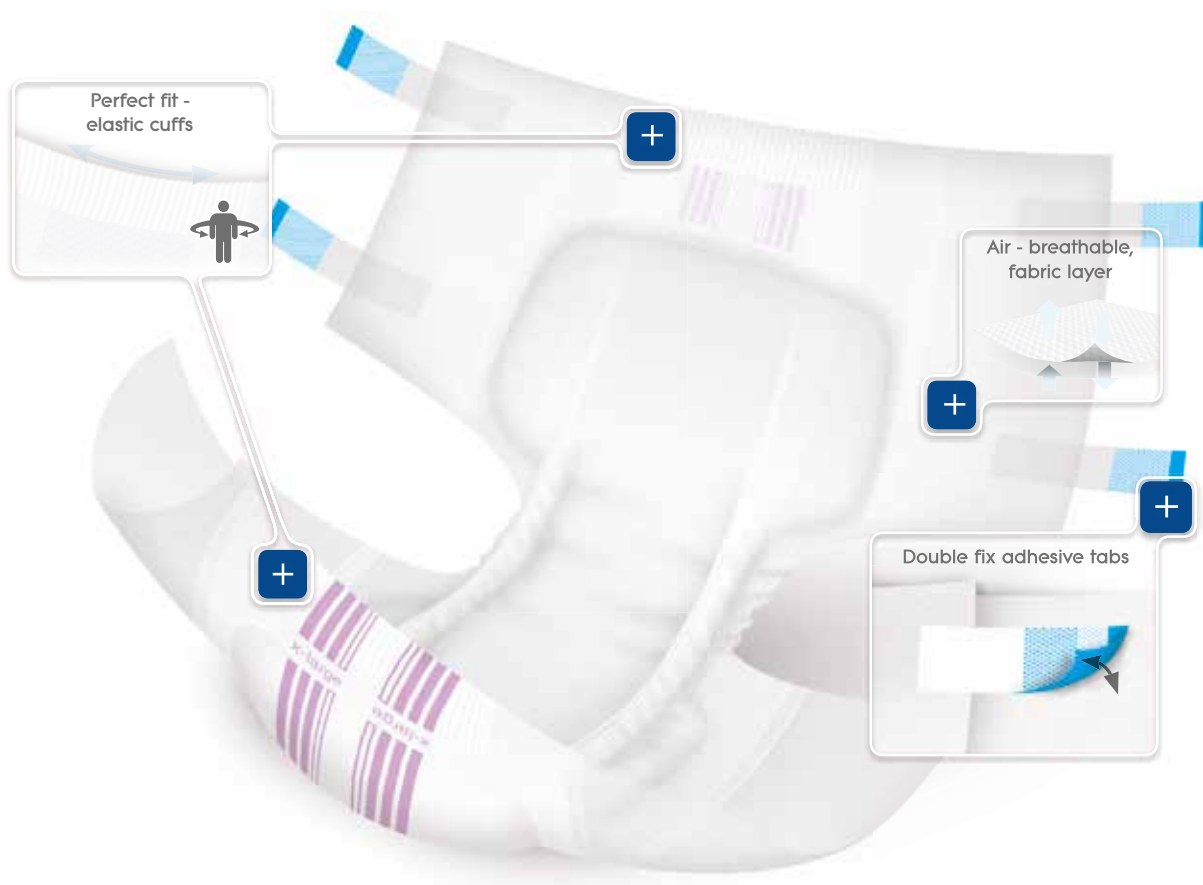

All in one slips



	NPC	MPC	EAN code	Brand name	code	Severity	Waist size	Weight in kg	Contents/ pack	ISO Absorbency in ml.
Sensitive	CFQ947	5155	4036978900291	forma-care slip Sensitive - plus s			50 - 90 cm	10	20	2100
	CFQ948	5065	4036978900307	forma-care slip Sensitive - super medium			70 - 125 cm	9	20	2480
	CFQ952	5562-01	4036978900314	forma-care slip Sensitive - extra medium			70 - 125 cm	11,1	18	3200
	CFP1987	5563-01	4036978013649	forma-care slip Sensitive - x plus medium			70 - 125 cm	10	18	3330
	CFQ949	5070-01	4036978900321	forma-care slip Sensitive - super large			100 - 155 cm	10,9	20	2500
	CFQ950	5567-01	4036978900338	forma-care slip Sensitive - extra large			100 - 155 cm	11,9	16	3800
	CFQ953	5567	4036978900338	forma-care slip Sensitive - extra large			100 - 155 cm	11,9	16	3950
	CFQ955	5568	4036978013663	forma-care slip Sensitive - extra plus large			100 - 155 cm	8,9	16	3950
	CFQ951	1124	4036978004791	forma-care slip PREMIUM - bariatric			160 - 190 cm	8,4	16	2980
	CFQ954	1124-01	4036978004791	forma-care slip PREMIUM - x large			115 - 175 cm	8,4	16	2980
Comfort	CFQ956	4156	4036978900369	forma-care slip comfort - small plus			50 - 90 cm	12	20	2790
	CFQ957	6065	4036978900178	forma-care slip comfort - medium plus			70 - 125 cm	8,9	20	2588
	CFQ958	9560	4036978900161	forma-care slip comfort - medium super			70 - 125 cm	10	20	3178
	CFQ961	9562	4036978900109	forma-care slip comfort - medium extra night			70 - 125 cm	13,9	18	3250
	CFP1989	9563-01	4036978900147	forma-care slip comfort - x plus medium			70 - 125 cm	11,5	18	4467
	CFQ959	6070	4036978900185	forma-care slip comfort - large plus			100 - 155 cm	10,9	20	3280
	CFP1986	6070-01	4036978900185	forma-care slip comfort - plus x large			115 - 175 cm	10,9	20	3280
	CFQ962	9565	4036978900192	forma-care slip comfort - large super			100 - 155 cm	14,9	20	4000
	CFQ965	9568	4036978900154	forma-care slip comfort - large x plus			100 - 155 cm	11,5	16	4790
	CFP1988	9568-01	4036978900154	forma-care slip comfort - x plus large			100 - 155 cm	11,5	16	4790
	CFQ960	4169	4036978900345	forma-care slip comfort - super extra large			115 - 175 cm	8,4	16	3890
	CFQ963	4169-01	4036978900345	forma-care slip comfort - super extra large			115 - 175 cm	8,4	16	3890
	CFQ964	2124	4036978900352	forma-care slip comfort - extra x large			115 - 175 cm	11	16	4790



How to fit whilst standing



Unfold the All-in-One and stretch out into the shape of a "ship". At the back of the product you will find the adhesive strips. Place the All-in-One from between your legs in the shape of a "ship" pulling through from front to back.




Smooth out the front side of the product. Unfold the back of the product - Check that the product is in the centre of your body, by lining up the wetness indicator with the spine.



When applying the front, fold the All-in-One sideways, do not pull up the product upwards. The lower adhesive strips fix horizontally, the upper adhesive strips, downwards and diagonally.

How to fit whilst lying down

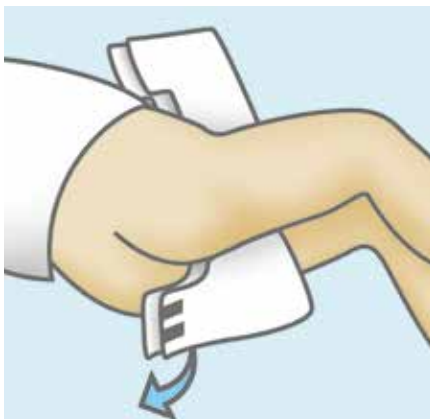

Unfold the All-in-One and stretch out into the shape of a "ship". At the back of the product you will find the adhesive strips. Place the All-in-One from between your legs in the shape of a "ship" pulling through from front to back.



Smooth out the front side of the product. Unfold the back of the product - Check that the product is in the centre of your body, by lining up the wetness indicator with the spine.



When applying the front, fold the All-in-One sideways, do not pull up the product upwards. The lower adhesive strips fix horizontally, the upper adhesive strips, downwards and diagonally.

User Instructions:

- Never shake or wring out the products.
- Only fold or stretch the products, if the instructions for folding the „ship“ are given.
- Do not allow contact with pointed objects when using the products – do not puncture.
- Do not work with dirty or greasy hands or when using care products. Make sure that no grease is caught on the outer layer, the adhesive strip or the inside layer of the product.