

Large shaped pads



Incontinence severity (involuntary loss of urine in 4 hours)



Highest quality of life with absolute reliability.

forma-care form is an anatomically shaped incontinence pad with ventilated TBS-air foil backing. The dermatologically tested and latex free product excels through its super absorbency qualities.



Large shaped pads




	NPC	MPC	EAN code	Brand name	Colour code	Severity	Size	Weight in kg	Contents/ pack	ISO Absorbency in ml.
Sensitive	CFP1959	5040	4036978900048	forma-care Sensitive form - plus			60 x 32 cm	6	20	1770
	CFP1960	5045	4036978900055	forma-care Sensitive form - super			70 x 35 cm	9	20	2138
	CFP1961	5050	4036978900062	forma-care Sensitive form - extra			70 x 35 cm	12	20	3384
Comfort	CFP1969	7040	4036978900239	forma-care comfort form - plus			60 x 32 cm	6	20	1770
	CFP1970	7040-01	4036978900239	forma-care comfort form - plus S			60 x 32 cm	6	20	1770
	CFP1971	7045	4036978900246	forma-care comfort form - super			70 x 35 cm	9	20	2138
	CFP1972	7050	4036978900253	forma-care comfort form - extra			70 x 35 cm	11	20	3384



Large shaped pads



How to fit whilst standing

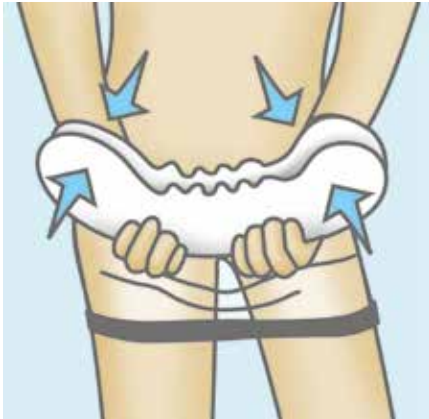

Raise the net fixation pants up to the middle of the thighs. Stretch the pad lengthways and fold along its length to form a small "ship" shape. The wider end fits to the back.




Pull the pad in the "ship" shape from front to rear, above the net fixation pants, through the legs. Unfold the front side of the pad but do not pull the product upwards. Check that the product is in the centre of your body, by lining up the wetness indicator with the spine.




Pull up the net fixation pants. Check the fit of the product, groin and crotch area. The folds of the thighs are positioned outside net the fixation pants.


How to fit Men's Shaped Pads

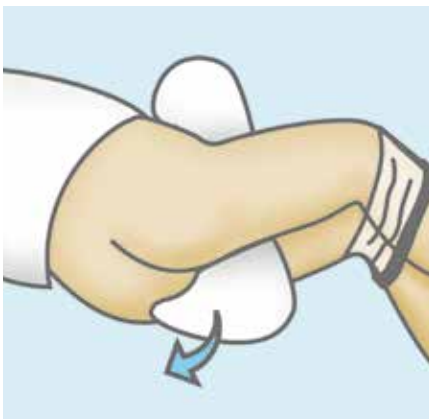

Lie on your side. Pull the fixation pants down below the knees. The wide side is the back. Draw the template in the form of a boat from front to back through the legs.

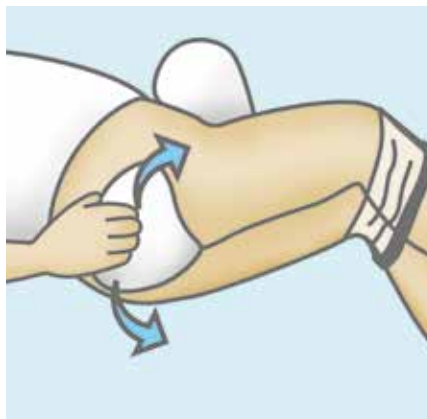

Pull the pad backwards slightly. First unfold the back, then the front. Do not pull the product upwards while doing this. Check that the product is in the centre of your body, by lining up the wetness indicator with the spine.

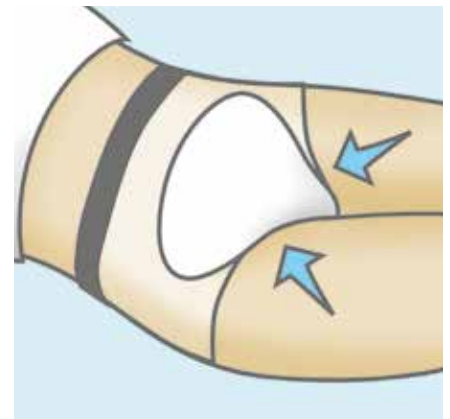

Pull the net fixation pants over the pad. Make sure that the fold of the skin on the thighs are outside the pad and net fixation pants. Check the fit again.

User Instructions:

- Never shake or wring out the products.
- Only fold or stretch the products, if the instructions for folding the „ship“ are given.
- Do not allow contact with pointed objects when using the products – do not puncture.
- Do not work with dirty or greasy hands or when using care products. Make sure that no grease is caught on the outer layer, the adhesive strip or the inside layer of the product.