

Pull up pants




Incontinence severity (involuntary loss of urine in 4 hours)



At home. In the park. In the restaurant and everywhere else.

Strong performance for an active life.

Easy to use, highest comfort and optimum safety provide an „A+“ quality of life.

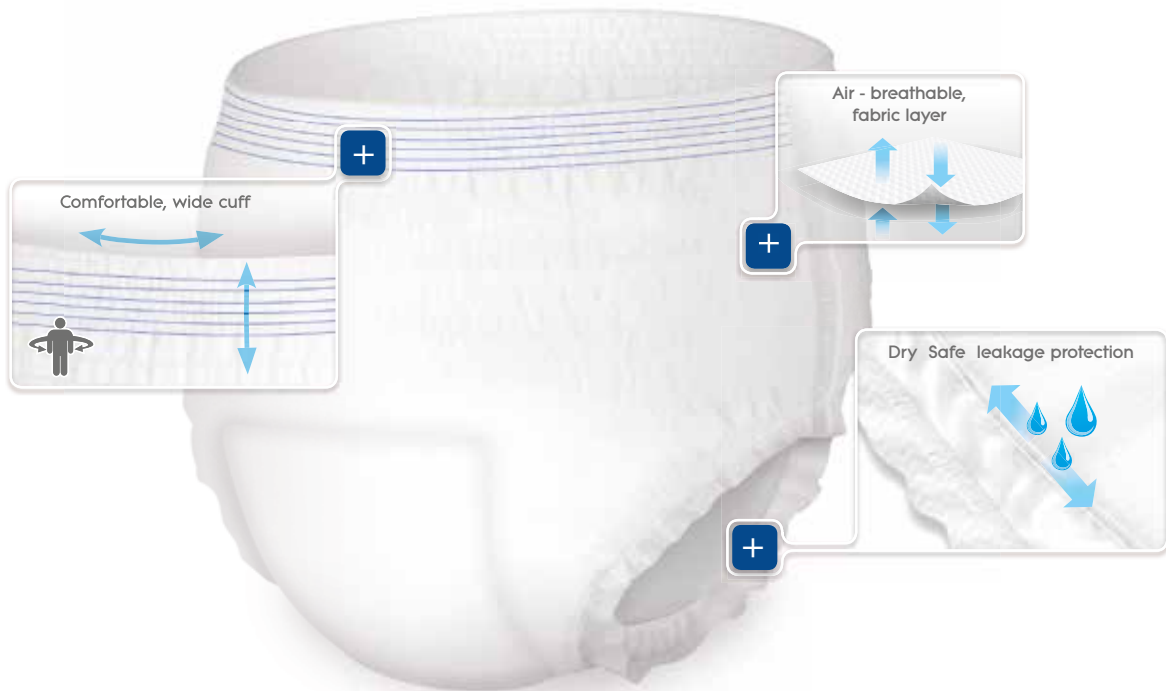

Pull up pants



PREMIUM

NPC	MPC	EAN code	Brand name	Colour code	Severity	Waist size	Weight in kg	Contents/ pack	ISO Absorbency in ml.
CFP1952	2071	4036978013274	forma-care pants PREMIUM - dry small			50 - 90 cm	5,5	14	1320
CFP1953	2074	4036978013250	forma-care pants PREMIUM - dry medium			75 - 120 cm	5,9	14	1460
CFP1956	2075	4036978013236	forma-care pants PREMIUM - dry medium night			75 - 120 cm	6,3	14	1800
CFP1954	2077	4036978013212	forma-care pants PREMIUM - dry large			100 - 155 cm	6,3	14	1590
CFP1955	2079	4036978013298	forma-care pants PREMIUM - dry x-large			120 - 175 cm	6,7	14	1700
CFP1957	2079-01	4036978013298	forma-care pants PREMIUM - dry x-large			120 - 175 cm	6,7	14	1700



How to fit whilst standing



Pull the pants up over the legs like under-pants.



Pull up the product so that it fits well into the groin and crotch area. Check the fit once again.



To take off the pants, rip open the side seams and dispose.

User Instructions:

- Never shake or wring out the products.
- Only fold or stretch the products, if the instructions for folding the „ship“ are given.
- Do not allow contact with pointed objects when using the products – do not puncture.
- Do not work with dirty or greasy hands or when using care products. Make sure that no grease is caught on the outer layer, the adhesive strip or the inside layer of the product.